PushTracker E2

Use the camera on your smartphone to scan QR codes.

During set-up, go to the Permobil website for more information and support:

Note: When upgrading from previous models, refer to the Permobil website for current product instruction and information.

① Get the Wear OS by Google app on your smartphone to set up your PushTracker E2:

② On your PushTracker E2, go to the Google Play Store and get:

SmartDrive MX2+

PushTracker

*** All figures shown are for illustrative purposes only and actual products may vary.*

*Google, Google Play and Wear OS by Google are trademarks of Google LLC.
Apple logo is a trademark of Apple, Inc.*

③ Swipe and touch to get around inside the SmartDrive MX2+ app for:

- Wireless Updates of your SmartDrive
- Adjusting your SmartDrive Settings

④ Customize your watchface with shortcuts to the apps or use Permobil's watchface:

⑤ Use your SmartDrive [MX2+ Control Mode]:

- Double tap to start driving the motor by ramping up its speed.
- Single tap to coast at the current speed.
- Double tap to deactivate the driving.
- Brake on the handrims to slow down.

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